



# ACCURO GUIDELINES

Epidural and Spinal Anesthesia Guidance with Accuro®

## DEVICE OVERVIEW

Accuro is superior to palpation for neuraxial anesthesia, with clinical benefits demonstrated by trainees, novice users, and experienced providers. Leverage automated, real-time image guidance to ensure first-attempt success, especially in challenging patient populations.

## PRO TIPS

- The learning curve is expected to take between 5 and 10 procedures.
- Initial device usage should occur with healthy subjects with normal BMI.
- Use slow, micro-movements when scanning with Accuro to allow the AI to interpret the ultrasound image as clear and concise as possible.
- Ensure a stable hand grip by holding Accuro near the transducer probe end with the same hand anchored to the patient. The other hand is positioned under Accuro for support.
- Remember your angle of insertion is dictated by the angle you scan with the device.

## LOCATOR & STERILE COVER KITS

The Accuro Locator and sterile cover with Locator needle guide kits are cleared by FDA as single use-only items and will not be re-used. Sterile technique is a requirement in the operating room or if re-scanning is desired after initial needle placement. The sterile cover and Locator are sold separately.

## RESOURCES

Visit [rivannamedical.com/accuro-resources](http://rivannamedical.com/accuro-resources)

- [Accuro Resources page](#)
- Accuro [IFU](#) & [FAQ's](#)
- Accuro Neuraxial Guidance - Scouting Technique: [Training Video](#)
- Accuro Neuraxial Guidance - Sterile Technique: [Training Video](#)
- Real World [Clinical Testimonials](#)
- [Accuro Academy](#) Tutorial (use code: 17043)